

MemoClear Pro e 3flex Clear sono fogli plastici per termoformatura individuale e solo per dispositivi dentali estetici prescritti esclusivamente da un professionista autorizzato e responsabile del loro impiego (vedi QR 1 video termoformatura).

Prima dell'uso accertarsi che il paziente non sia allergico a materie plastiche.

Ulteriori informazioni sono contenute nelle schede tecniche MemoClear PRO (QR 2) e 3flex Clear (QR 3)

Conservare i fogli nella loro confezione lontani da fonti di calore, umidità e al riparo dalla luce.

Per 3flex Clear, richiudere immediatamente la confezione dopo il prelievo dei fogli.

Rifinire i dispositivi con strumenti adeguati, per evitare spigoli vivi, incrinature ecc...(vedi QR4 video rifinitura).

MemoClear Pro and 3flex Clear are plastic sheets for individual thermoforming and for aesthetic dental devices only, exclusively prescribed by a licensed professional and responsible for their use (see QR 1 thermoforming video).

Before use, ensure that the patient is not allergic to plastic materials.

Further information is contained in the MemoClear PRO (QR 2) and 3flex Clear (QR 3) technical data sheets.

Store the sheets inside their packaging away from heat, humidity, and light.

For 3flex Clear, immediately reseal the packaging after removing the sheets.

Finish the devices with appropriate tools, to avoid any sharp edges, cracks, etc. (see QR4 finishing video).



QR 1



QR 2



QR 3



QR 4

MemoClear Pro

Tipo & Codice Type & Code			Spessori Thicknesses		Riscaldamento Heating		Raffredd. Cooling	Biostar/ Ministar
Quadrato Square 127x127mm	Tondo Round 120mm Ø	Tondo Round 125mm Ø	mm / inch		Temp.	Sec. time	Sec. time	Codici Code
TP-3441	TP-3449	TP-3431	0.50	0.020	160°C	25 sec	20 sec	101
TP-3442	TP-3450	TP-3432	0.625	0.025	160°C	25 sec	60 sec	101
TP-3443	TP-3439	TP-3433	0.75	0.030	160°C	30 sec	60 sec	112
TP-3444	TP-3451	TP-3434	0.85	0.035	160°C	30 sec	60 sec	112
TP-3445	TP-3440	TP-3435	1.0	0.040	160°C	35 sec	120 sec	122
TP-3447	TP-3452	TP-3437	1.5	0.060	160°C	40 sec	120 sec	143
TP-3448		TP-3438	2.0	0.080	160°C	45 sec	120 sec	163

3flex Clear

Tipo & Codice Type & Code		Spessori Thicknesses		Riscaldam. Heating		Raffredd. Cooling	Biostar/ Ministar
Tondo Round 120mm Ø	Tondo Round 125mm Ø	mm / inch		Temp.	Sec. time	Sec. time	Codici Code
TP3-120-50	TP3-125-50	0.80	0.031	160°C	30 sec	60 sec	112

Come rimuovere la pellicola protettiva dopo la finitura dell'allineatore
How to remove the protective film after finishing the aligner

